

EVERY DROP COUNTS

Reduce blood loss with the OKAY® II Y-Connector

NIPRO
OKAY® II

Y-Connector

NOW AVAILABLE IN THE USA
#1 selling Y-Connector in Japan

Key Benefits

- 1** 10Fr Inner Lumen Diameter
Suitable for a variety of advanced multi-device techniques.
- 2** One-Touch Control
The hemostatic valve is easy-to-use with single hand operation.
- 3** Minimize Blood Loss
Blood loss during procedure can be significantly reduced with the OKAY® II device compared with other products without the hemostatic valve.
- 4** Hemostatic Valve with Convenient Lock-Open Feature
Once the hemostatic valve has been opened, a 1/4 turn locks the valve in the open position.
- 5** The Fixed Valve
The fixed valve on the OKAY® II device can be closed by turning the thumb wheel clockwise with a "1 revolution" motion. The fixed valve secures devices in place during contrast injection, ballooning or stenting.
- 6** Prevent Loss of Pressure
OKAY® II pressure resistance with fixed valve closed $\leq 500\text{psi}^*$

*Measurements were obtained without any devices inserted. Use of devices (including multiple devices) may result in instances whereby 500psi is not achieved.




Get the OKAY® II in your hands today.

Request Information at

info@nipromed.com | www.nipro.com | +1.908.393.7030

NIPRO

Valve Operation

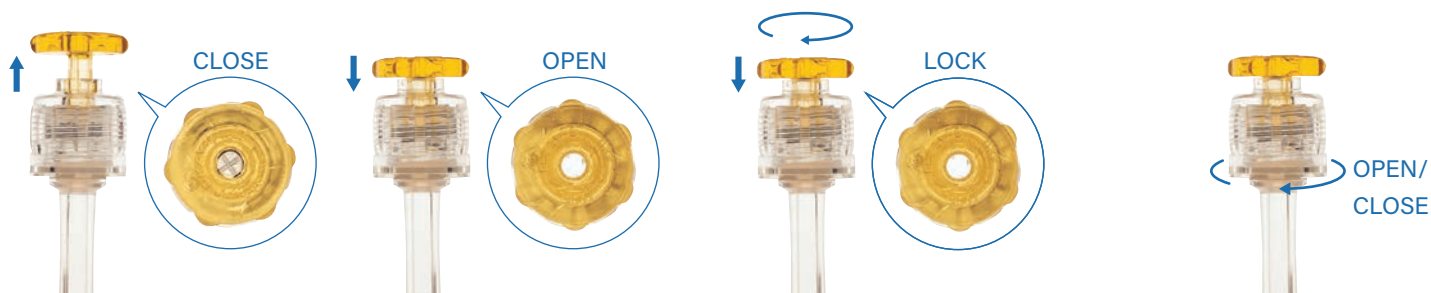

Hemostatic Valve

The valve can be opened by pushing the hemostatic valve stem, and closed by releasing it, with a single hand operation.

Once the hemostatic valve has been opened, a clockwise quarter turn locks the valve in the open position


Fixed Valve

Valve to secure devices can be closed with one revolution

Accessories

- ① **OKAY® II Y-Connector**
10Fr inner lumen
≤500psi, with fixed valve closed*
- ② **Torque Device**
Compatible Guidewire Diameters: 0.010"-0.018"
- ③ **Insertor**
Inner Diameter = 0.021" (max guidewire size: 0.018")
- ④ **Extension Tube**
22cm, Pressure Resistance ≤500psi

*Measurements were obtained without any devices inserted. Use of devices (including multiple devices) may result in instances whereby 500psi is not achieved.

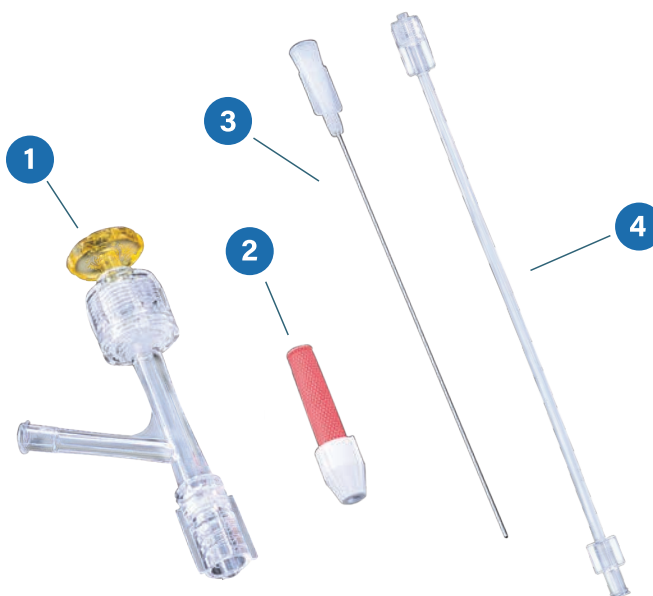

Ordering Information

Product Name	Product Reference	Contents	GTIN
OKAY® II Y-Connector Set	YOK0A	Y-Connector, Insertor, Torque Device (qty 5)	04543660022331
OKAY® II Y-Connector	YOK0B	Y-Connector (qty 5)	04543660022348
OKAY® II Y-Connector Set w Ext. Tube	YOK0E	Y-Connector, Insertor, Torque Device, Extension Tube (qty 5)	04543660022355
GM-30 Accessory Kit	GLGMN-YOK0A	A single box containing 1 sterile pouch with OKAY® II Y-Connector Set and 1 sterile pouch with a GM-30N Inflation Device.	04543660023508
GM-30 Accessory Kit	GLGMN-YOK0E	A single box containing 1 sterile pouch with OKAY® II Y-Connector Set w Ext. Tube and 1 sterile pouch with a GM-30N Inflation Device.	04543660023515
GM-30 Accessory Kit	GLGMF-YOK0A	A single box containing 1 sterile pouch with OKAY® II Y-Connector Set and 1 sterile pouch with a GM-30NF Inflation Device.	04543660023485
GM-30 Accessory Kit	GLGMF-YOK0E	A single box containing 1 sterile pouch with OKAY® II Y-Connector Set w Ext. Tube and 1 sterile pouch with a GM-30NF Inflation Device.	04543660023492

Distributed by



Nipro Medical Corporation
200 Crossing Boulevard
Bridgewater, NJ 08807
USA

T: +1.908.393.7030

Request Information at

info@nipromed.com
www.nipro.com

Imported by

Infraredx, a Nipro Company
28 Crosby Drive, Suite 100
Bedford, MA 01730
USA

T: +1.888.680.REDX (7339)
www.infraredx.com

©2022 Infraredx™, a Nipro Company
All rights reserved. MK0016rD